

Viscotears[®] HA

Preservative free eye drops containing 0.1% sodium hyaluronate
10ml Multi-dose bottle

CAREFULLY READ THIS LEAFLET BEFORE FIRST USE

Viscotears[®] HA is a sterile, ophthalmic solution with a soothing and moisturising effect and a physiological pH. It is based on sodium hyaluronate.

Sodium hyaluronate is a viscous polymer commonly used for its moisturising and lubricating properties. It creates a protective, transparent, viscoelastic film on the ocular surface. Viscotears[®] HA decreases the symptoms of mild to moderate dry eye such as redness, burning, irritation and foreign-body sensation. These symptoms are often associated with contact lens wear, the use of computer or device screens, certain medical conditions and treatments, environmental factors such as air-conditioning or central heating and atmospheric pollution.

Thanks to the OSD closure system, which maintains the microbiological, chemical and physical characteristics, Viscotears[®] HA does not contain any preservatives. This solution is compatible with contact lenses.

INSTRUCTIONS FOR USE

- Tear off the tamper-evident seal before the first use.
- Open the bottle by removing the protective cap.
- Apply 1-2 drops in each eye by gently squeezing the bottle.
- After use, tip the bottle downwards and shake of any excess product before replacing the protective cap. Make sure not to touch the applicator with your fingers or any other surface.
- The contents should be used within 6 months from the date of first opening.
- Suitable for frequent use: Unless instructed otherwise by your doctor or pharmacist, use these drops 2-3 times per day or as often as is required. As dry eye is a condition which varies from person to person, the drops should be used as often as you think necessary, to relieve your dry eye.

As the bottle empties during the course of normal use, you may need to apply slightly more pressure when you squeeze the bottle to deliver the drops. This is due to the OSD closure system that is used to preserve the sterility of the preservative free solution.

INGREDIENTS

0.1% sodium hyaluronate, sodium phosphate monobasic monohydrate, sodium phosphate dibasic dodecahydrate, sodium chloride, W.F.I.

Free from preservatives.

BAUSCH + LOMB

WARNINGS AND PRECAUTIONS

- The product is for single patient use.
- The product is only intended for ophthalmic use.
- Do not use this product if you are hypersensitive to any of the ingredients.
- Do not use the product if, at the time of first opening, the product appears damaged or the tamper evident seal is broken.
- Apply the product before inserting contact lenses.
- The administration of ophthalmic drugs concurrently with these eye drops should only be carried out after discussion with your doctor who can evaluate the possible interactions and should be carried out at least 15 minutes after Viscotears HA eye drops are instilled.
- During application, do not touch your eyes or any other surface with the tip of the bottle.
- If irritation occurs, stop using the product and speak to your doctor.
- Do not use after the expiry date stated on the carton and multi-dose bottle.
- Dispose of the container after use following local regulations.
- Keep out of reach of children.
- Do not swallow.
- The product is usually intended for continuous use for no more than 30 days; consult your doctor if symptoms persist or worsen.
- If you are pregnant or breastfeeding, consult your doctor or health care professional before use.

COMMON SIDE EFFECTS

- Minor burning/stinging/irritation may occur temporarily as a result of existing ocular irritation. If symptoms persist or worsen, contact your doctor.

PRESENTATION

10ml multi-dose bottle, preservative free.

STORAGE CONDITIONS

Do not store above 30°C.
Discard 6 months after first opening.



C.O.C. Farmaceutici S.r.l.
Via Modena, 15
40019 - Sant'Agata
Bolognese (Bo)
Italy

MEDICAL DEVICE

CE 0546

DISTRIBUTED BY:

Bausch & Lomb UK Ltd
106 London Road
Kingston upon Thames
KT2 6TN

SYMBOLS



Sterilised using aseptic processing techniques



Upper temperature limit



Do not use if packaging is damaged



Consult instructions for use



Caution: Consult the "instructions for use" for important cautionary information such as warnings and precautions



Use by date



Batch code



Manufacturer

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates

© Bausch & Lomb Incorporated

Last Revision Date 04.02 of 10/2021 BLv2

BAUSCH + LOMB