

# Viscotears® Trehal Duo

**Preservative free eye drops containing 2.0% trehalose and 0.25% sodium hyaluronate**

10ml Multi-dose bottle

## **CAREFULLY READ THIS LEAFLET BEFORE FIRST USE**

Viscotears® Trehal Duo is an intensive moisturising eye drop intended to relieve moderate and severe dry eye symptoms. It contains trehalose and sodium hyaluronate.

The combined action of trehalose and sodium hyaluronate provides hydration, lubrication and stabilisation of the tear film, protecting the eye from drying and cell damage. Viscotears® Trehal Duo lowers tear film osmolarity, helping the ocular surface recover from damage and redness. Regular use relieves dry eye symptoms such as itching, burning and foreign body sensation caused by environmental factors such as dry air, wind, smoke, air pollution, excessive light, visual display work, ocular surgery or contact lens use.

This sterile, preservative free eye drop is suitable for those with sensitive eyes or contact lens wearers and is appropriate for long-term and frequent use.

## **INSTRUCTIONS FOR USE**

- Practice handling of the bottle before the first application
- Tear off the tamper-evident seal before first use
- Open the bottle by removing the protective cap
- Squeeze the bottle only with the tip down
- Apply 1-2 drops in to each eye, 1-3 times a day as needed
- Shake any remaining solution from the tip and replace the protective cap immediately after use
- You can use Viscotears® Trehal Duo whilst wearing contact lenses

## **INGREDIENTS**

2.0 % trehalose, 0.25 % sodium hyaluronate, disodium hydrogen phosphate dodecahydrate, purified water, sodium chloride, sodium dihydrogen phosphate dihydrate.

**Free from preservatives.**

**Free from materials of animal origin.**

**BAUSCH + LOMB**

## WARNINGS AND CAUTIONS

- Do not use if you are sensitive to any of the ingredients.
- Do not use if the bottle is damaged.
- Each bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with your fingers.
- Do not squeeze the bottle with the cap on.
- Do not use after the expiry date or 6 months after first opening – whichever is sooner.
- Mild temporary blurring of vision may be experienced immediately after application. It is therefore advisable to wait a few seconds before continuing any activity that requires clear vision.
- Leave at least 10-15 minutes between applications of any other ophthalmic preparation.
- Keep out of the reach and sight of children.
- Store at 15-30 °C, protect from direct sunshine and freezing.
- Discard the empty bottle appropriately.

## PRESENTATION

10ml multi-dose bottle, preservative free.

## STORAGE CONDITIONS

Store at 15-30 °C, protect from sunlight and freezing.

Discard 6 months after first opening.

 Oy Finnsusp Ab,  
Pääskykalliontie 5,  
21420 Lieto, Finland.

## MEDICAL DEVICE

 0537

## DISTRIBUTED BY:

Bausch & Lomb U.K. Ltd.  
106 London Road  
Kingston upon Thames  
KT2 6TN  
United Kingdom

2571-1

## SYMBOLS



Sterilised using aseptic processing techniques



Temperature limits



Do not use if packaging is damaged



Consult instructions for use



Caution: Consult the "instructions for use" for important cautionary information such as warnings and precautions



Use by date



Batch code



Manufacturer



Single patient, multiple use



Keep out of reach of children



Protect from direct sunlight



Discard 6 months after first opening

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates

© Bausch & Lomb Incorporated

Latest revision: 03/2021, v. 2.3-V

**BAUSCH + LOMB**