

Viscotears[®] Tri Action

Preservative free eye drops containing 2.0% trehalose, 0.2% sodium hyaluronate, 0.1% sacha inchi seed oil and glycerol

10ml Multi-dose bottle

CAREFULLY READ THIS LEAFLET BEFORE FIRST USE

Viscotears[®] Tri Action is a lipid microemulsion eye drop for the treatment and prevention of signs and symptoms of moderate or severe dry eye, especially where lipid supplementation is required.

Tear film instability can lead to excessive evaporation of the tear film which, in turn, leads to hyper osmolarity of the tear film (where the tear film becomes too salty). Tear film instability and hyper osmolarity can induce ocular surface inflammation and damage.

Viscotears[®] Tri Action has been designed to counteract all major factors that contribute to dry eye.

- Sodium hyaluronate, trehalose and glycerol bind water in the tear film and, with the mildly hypotonic (low osmolarity) solution, bring tear film osmolarity to a safe level allowing the ocular surface to recover from damage.
- Sacha inchi seed oil strengthens the tear film's lipid layer to prevent further evaporation and hyper osmolarity.
- The combination of oil and high-molecular-weight sodium hyaluronate provides extended retention and stabilisation of the tear film's lipid and muco-aqueous layers.
- Sodium hyaluronate, trehalose and natural antioxidants of sacha inchi seed oil assist ocular cells in recovery from inflammatory damage.

Regular use relieves dry eye symptoms such as itching, burning and foreign body sensation caused by environmental factors such as dry air, wind, smoke, air pollution, excessive light, visual display work, ocular surgery or contact lens use.

This sterile, preservative free eye drop is suitable for contact lens wearers.

INSTRUCTIONS FOR USE

- Practice handling of the bottle before the first application
- Tear off the tamper-evident seal before first use
- Open the bottle by removing the protective cap
- Squeeze the bottle only with the tip down
- Apply 1 drop in to each eye, 3-4 times a day.
- Shake any remaining solution from the tip and replace the protective cap immediately after use
- You can use Viscotears[®] Tri Action whilst wearing contact lenses

INGREDIENTS

Trehalose 2%, sodium hyaluronate 0.2%, sacha inchi seed oil 0.1%, glycerol, citric acid monohydrate, polyoxyethylenesorbitan monooleate, potassium chloride, purified water, sorbitan monooleate, tromethamine.

Free from preservatives, alcohol, phosphates, borates, mineral oil, or materials of animal origin.

BAUSCH + LOMB

WARNINGS AND CAUTIONS

- Do not use if you are sensitive to any of the ingredients.
- Do not use if the bottle is damaged.
- Each bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with your fingers.
- Do not squeeze the bottle with the cap on.
- Do not use after the expiry date or 6 months after first opening – whichever is sooner.
- If you experience blurring of vision just after application, please wait a moment before continuing activities that require clear vision. Blurring should be mild and temporary only.
- Wait at least 15 minutes between possible applications of other ophthalmic medication.
- Keep out of the reach and sight of children.
- Store at 5–30 °C, protect from direct sunshine and freezing.
- Discard the empty bottle appropriately.

PRESENTATION

10ml multi-dose bottle, preservative free.

STORAGE CONDITIONS

Store at 5–30 °C, protect from sunlight and freezing.

Discard 6 months after first opening.

 Oy Finnsusp Ab,
Pääskykalliontie 5, 21420
Lieto, Finland.

Produced using proprietary DEMET™ microemulsion technology developed by Oy Finnsusp Ab

MEDICAL DEVICE

 0537

DISTRIBUTED BY:

Bausch & Lomb U.K. Ltd.
106 London Road
Kingston upon Thames
KT2 6TN
United Kingdom

2573-1

SYMBOLS



Sterilised using aseptic processing techniques



Temperature limits



Do not use if packaging is damaged



Consult instructions for use



Caution: Consult the “instructions for use” for important cautionary information such as warnings and precautions



Use by date



Batch code



Manufacturer



Single patient, multiple use



Keep out of reach of children



Protect from direct sunlight



Discard 6 months after first opening

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates

© Bausch & Lomb Incorporated

Last revision: March 2021, v. 1.1-V

BAUSCH + LOMB